

# Preference Card

**The following supplies are needed for optimal results:**

## **Local:**

Topical Anesthetic Gel/Spray- (used for localization of mucosa inside mouth)  
(20% Benzocaine Oral Anesthetic)

1% Lidocaine with Epinephrine

8.4% Sodium Bicarbonate- (used to buffer the acidity of local)

10cc syringe

30g needle 1” long

## **Prep:**

Betadine

**Procedure: (we recommend the lip procedure be performed under sterile conditions)**

Sterile OR Towels x 4

Sterile Gloves

Sterile 4x4s

4-0 absorbable suture (chromic P-3)

## **Instruments:**

Perma Tunneler

Brown Adson Forceps

#15 blade or curved Iris Scissors to make incisions

Curved Iris Scissors to use for dissection

Needle Holder

## **Dressings:**

Bacitracin Ointment (to be applied topically to lips and incisions after procedure)

**Disclaimer: The intended use of the PermaLip™ Product is not approved for distribution within the United States.**