

Anesthesia for Lip Augmentation Procedures:

Spray hurricane or apply anesthetic gel to the buccal mucosa.

Wait 2-3 minutes.

Draw up 10-12cc of Lidocaine with epinephrine mixed with 8.4% bicarbonate

Inject 1.5-2cc of local into the infraorbital and mental nerves through the mucosa that has been numbed rather than going through the skin (unless you've chosen to freeze the skin prior to injection).

Also inject the upper frenulum area to lessen the pain attributable to the inferior alveolar nerves.

Wait 5-10 minutes.

Inject local into the lips directly starting with the commissure and going toward the midline. (We choose to use a 30ga. Needle on a 10 or 12cc syringe.) Be sure to inject the local into the lips along the wet-dry border in the deep submucosal plane, using about 2ccs per lip.

Wait 5-10 minutes to start the procedure or when blanching appears from the epinephrine.

Disclaimer: The intended use of the PermaLip Product™ is not approved for distribution within the United States.